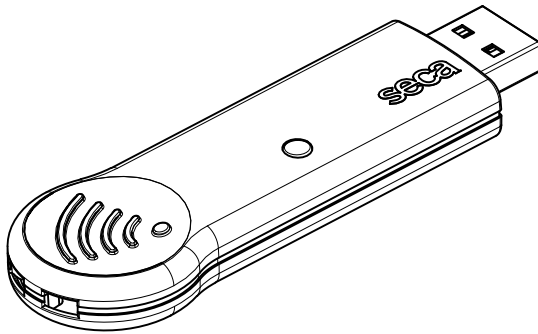
seca 456 WA

| | | |
|-----------|---|-----------|
| DE | Gebrauchsanweisung und Garantieerklärung | 3 |
| EN | Instruction manual and guarantee | 9 |
| FR | Mode d'emploi et garantie | 14 |
| IT | Manuale di istruzioni e garanzia | 20 |
| ES | Manual de instrucciones y garantía | 26 |
| DK | Betjeningsvejledning og garantibevis | 32 |
| SE | Bruksanvisning och garanti | 37 |
| NO | Bruksanvisning og garantierklæring | 42 |
| FI | Käyttöohje ja takuu | 47 |
| NL | Bedieningshandleiding en garantieverklaring | 52 |
| PT | Instruções de utilização e declaração de garantia | 57 |
| GR | Οδηγίες χειρισμού και εγγύηση | 63 |
| PL | Instrukcja obsługi i gwarancja | 69 |
| RU | Инструкция по эксплуатации и гарантийное свидетельство | 75 |
| JP | 使用説明書および保証 | 81 |
| CN | 说明书和保证 | 86 |
| | Declaration of conformity for USA and Canada | 91 |

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1. INTENDED USE

The **seca 456 WA** is mainly used in hospitals, medical practices and inpatient care facilities in accordance with national regulations.

The accessory part **seca 456 WA** is solely for transmitting data between **seca 360° wireless** devices and compatible devices made by the manufacturer Welch Allyn. The accessory part **seca 456 WA** is not compatible with seca PC software.

The accessory part **seca 456** is obtainable for operating **seca 360° wireless** devices with seca PC software.

2. SAFETY INFORMATION

- ▶ Follow the instructions for use.
- ▶ Follow the instructions for use for your **seca 360° wireless** devices and the Welch Allyn device used.
- ▶ Keep the instructions for use in a safe place.
- ▶ To prevent malfunctions, do not operate or store the device in an environment in which it is exposed to high humidity, vibrations, dust, heat or direct sunlight.
- ▶ Never immerse the device in disinfectant or other liquids.
- ▶ Do not start up the device immediately after transporting it. Severe temperature fluctuations and changes in humidity may lead to moisture forming on the device. This may damage both the device itself and the Welch Allyn device to which it is connected.
- ▶ Secure the device to stop it being removed from the Welch Allyn device inadvertently; this will prevent malfunctions and use not in accordance with purpose.
- ▶ Keep other medical electrical devices a minimum distance of approx. 1 meter away to prevent incorrect measurements or wireless transmission interference.
- ▶ Keep HF equipment such as cell phones and screens a minimum distance of approx. 1 meter away to prevent incorrect measurement or wireless transmission interference.

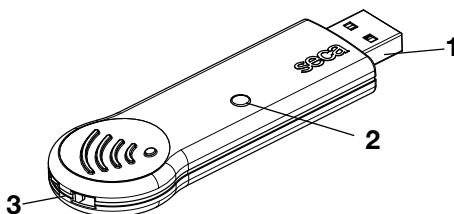
- ▶ The actual transmission output of HF equipment may require minimum distances of more than 1 meter. For details, go to www.seca.com.
- ▶ If the device has to be operated in the immediate vicinity of individual or stacked medical electrical devices, ensure that the devices are working properly.
- ▶ Do not use extension cables. These may impair the EMC immunity of the device.
- ▶ Do not open the device housing. This can lead to damage to the device. seca does not provide any kind of warranty in such cases.

3. OVERVIEW

3.1 Scope of delivery

| Component | Pcs. |
|--|------|
| USB adapter | 1 |
| Extension cable (article number: 08-06-16-489) | 1 |
| Hook-and-loop tape (article number: 50-90-01-327) | 1 |
| Instructions for use with "Authorization Code" sticker | 1 |

3.2 Controls

| No. | Control | Description |
|-----|---------------|---|
| 1 | USB connector | Is plugged into a free USB port on the Welch Allyn device |
| 2 | LED | <ul style="list-style-type: none"> • Green: USB adapter correctly detected by Welch Allyn device • Blue: data being transmitted |
| 3 | Securing lug | To stop the USB adapter being removed from the Welch Allyn device inadvertently |

3.3 Information on the device and the type plate

| Text/symbol | Meaning |
|---|---|
|  | Model number |
|  | Serial number, consecutive |
|  | Follow instructions for use |
|  | Device complies with EU standards and directives |
|  | Device is equipped with a seca wireless module, data is received and transmitted wirelessly |
|  | Symbol of the US Federal Communications Commission (FCC) |
| FCC ID | Device approval number from the Federal Communications Commission (FCC) |
| IC | Device license number from Industry Canada |
|  | Do not dispose of device with household waste |

3.4 Marking on the packaging

| | |
|---|---|
|  | Permitted min. and max. temperature for transport and storage |
|  | Permitted min. and max. moisture for transport and storage |

4. OPERATION

The **seca 456 WA** USB adapter is for transmitting data between **seca 360° wireless** devices and compatible devices made by the manufacturer Welch Allyn. These instructions for use describe only operation of the **seca 456 WA** USB adapter. For information about operating your Welch Allyn device, go to www.welchallyn.com.

1. Plug the USB adapter into a free USB port on your Welch Allyn device.

2. Secure the USB adapter to stop it being removed from the Welch Allyn device inadvertently.
3. Set up a **seca 360° wireless** network as described in the instructions for use for your **seca 360° wireless** device.
4. If the instructions for use for your **seca 360° wireless** device request you to switch on all the components of a wireless group, switch on the Welch Allyn device as well.

NOTE

The Welch Allyn device is shown in the display of the **seca 360° wireless** device as module "MO 4".

5. Continue setting up the wireless network as described in the instructions for use for your **seca 360° wireless** device.

5. HYGIENE TREATMENT

- ▶ Use a soft cloth dampened with mild soapsuds to clean the surfaces of the device.
- ▶ Disinfect the device as required with a commercially available disinfectant, e.g. with 70 % isopropyl alcohol.
- ▶ The device may not be sterilized.

6. TECHNICAL DATA

| Technical data seca 456 WA | |
|---|--|
| Measurements <ul style="list-style-type: none"> • Depth • Width • Height | approx. 27 mm approx. 85 mm approx. 10 mm |
| Net weight | 0.01 kg |
| Ambient conditions, operation <ul style="list-style-type: none"> • Temperature • Air pressure • Humidity | +10 °C to +40 °C (50 °F to 104 °F) 700 hPa - 1060 hPa 30 % - 80 %, no condensation |
| Ambient conditions, storage <ul style="list-style-type: none"> • Temperature • Air pressure • Humidity | -10 °C to +65 °C (14 °F to 149 °F) 700 hPa - 1060 hPa 0 % - 95 %, no condensation |

| Technical data seca 456 WA | |
|--|---|
| Ambient conditions, transport <ul style="list-style-type: none"> • Temperature • Air pressure • Humidity | -10 °C to +65 °C (14 °F to 149 °F) 700 hPa - 1060 hPa 0 % - 95 %, no condensation |
| Power supply | 5 V via PC's USB slot |
| Wireless transmission <ul style="list-style-type: none"> • Frequency band • Transmission power • Channel spacing • Channels • Data rate | 2,433 Ghz -2,480 GHz < 10 mW 199,951 kHz 0-99 50 kBaud |

7. DISPOSAL

Do not dispose of the device with household waste. The device must be disposed of properly as electronic waste. Comply with the national provisions applicable in your country. For further information contact our service department at:

service@seca.com

8. GUARANTEE

We offer a two-year warranty from the date of delivery for defects attributable to faulty material or poor workmanship.

9. DECLARATION OF CONFORMITY

seca gmbh & co. kg hereby declares that the product meets the terms of the applicable European directives. The unabridged declaration of conformity can be found at: www.seca.com.

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DECLARATION OF CONFORMITY FOR USA AND CANADA

seca
seca 456 WA

FCC ID: X6T172A01

IC: 8898A-172A01

This device complies with Part 15 of the FCC Rules and with RSS-210 of Industry Canada. Operation is subject to the following two conditions. (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

NOTE

This device complies with Part 15 of the FCC Rules and with RSS-210 of Industry Canada. Operation is subject to the following two conditions:

- This device may not cause harmful interference.
- This device must accept any interference received, including interference that may cause undesired operation.

NOTE

Changes or modifications made to this equipment not expressly approved by seca may void the FCC authorization to operate this equipment.

NOTE

Radiofrequency radiation exposure Information:

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This equipment should be installed and operated with minimum distance of 1 m between the radiator and your body. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

Medical Measuring Systems and Scales since 1840

seca gmbh & co. kg
Hammer Steindamm 3–25
22089 Hamburg · Germany
Telephone +49 40 20 00 00 0
Fax +49 40 20 00 00 50
info@seca.com

seca operates worldwide with headquarters
in Germany and branches in:

seca france

seca united kingdom

seca north america

seca schweiz

seca zhong guo

seca nihon

seca mexico

seca austria

seca polska

seca middle east

seca brasil

seca suomi

seca américa latina

seca asia pacific

and with exclusive partners in
more than 110 countries.

All contact data under www.seca.com

seca[®]
Precision for health