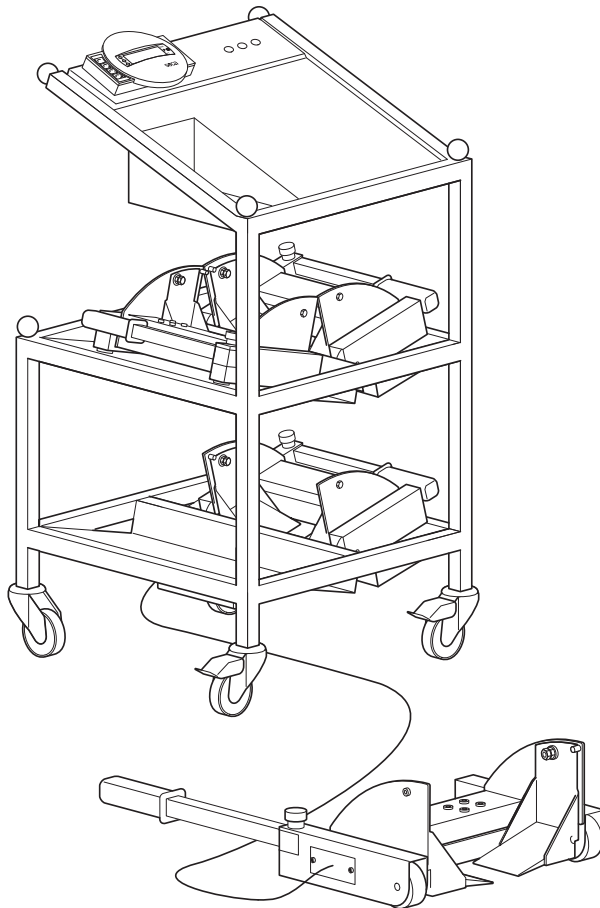
seca 985

ⓓ	<i>Bedienungsanleitung und Garantieerklärung</i>	3
ⓖⓑ	<i>Instruction manual and guarantee</i>	21
ⓕ	<i>Mode d'emploi et garantie</i>	42
ⓓ	<i>Manuale di istruzioni e garanzia</i>	57
ⓔ	<i>Manual de instrucciones y garantía</i>	75
ⓓⓀ	<i>Betjeningsvejledning og garantibevis</i>	93
Ⓢ	<i>Bruksanvisning och garanti</i>	111
Ⓝ	<i>Bruksanvisning og garantierklæring</i>	129
ⓕⓘⓃ	<i>Käyttöohje ja takuu</i>	147
Ⓝⓕ	<i>Bedieningshandleiding en garantieverklaring</i>	165
ⓑ	<i>Instruções de utilização e declaração de garantia</i>	183
ⓖⓚ	<i>Οδηγίες χειρισμού και εγγύηση</i>	201
ⓑⓕ	<i>Instrukcja obsługi i gwarancja</i>	219

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Full certification

seca products you are purchasing not only technology matured over a century, but also quality confirmed by authorities, the legal system and by institutes.

seca products comply with European directives, standards and national laws. With seca, you are buying the future.

The products in these operating instructions comply with German legislation governing medical products and therefore with Directive 93/42/EEC of the Council of the European Community which is embodied in national laws throughout Europe.

Scales carrying this symbol are calibrated in accordance with Precision Class III of EC Directive 2014/31/EU.

Device conforms to EC directives

- **M**: Conformity label according to Directive 2014/31/EU governing non-automatic weighing instruments (verified models)
- **18**: (Example: 2018) Year in which conformity verification was performed and the CE label was applied (verified model)
- **0102**: Notified body metrology (verified models)
- **0123**: Notified body medical products

seca's professionalism has also been officially recognized. The certificate from TÜV Product Service, the body responsible for medical devices, confirms that as a manufacturer of medical devices, seca rigorously complies with stringent legal requirements. seca's quality assurance system covers the design, development, production, sales and service of medical scales and measuring systems.

seca helps the environment.

We are concerned to save natural resources. We therefore strive to save packaging material where meaningful. What is left over can conveniently be disposed of locally via the German Dual System (recycling programme).

1. Congratulations!

In the **seca 985** electronic bed scale you have acquired a piece of high-precision equipment which allows bed-ridden patients to be weighed simply and gently.

seca has been putting its experience at the service of health for over 170 years now, and as market leader in many countries of the world, is always setting new standards with its innovative developments for weighing and measuring.

The **seca 985** electronic bed scale is used primarily in hospitals and dialysis centres in accordance with national regulations. The scale has Class III CE approval.

In addition to the mobile bed scale with its equipment trolley, there is also a stationary model in which the measuring electronics are accommodated in a housing underneath the bed. These instructions describe the mobile bed scale but apply equally to the stationary model.

The new type of lifting mechanism allows

the scale to be installed with minimum effort. It is no longer necessary to move the bed, and the swivel position of the bed's castors is irrelevant.

Weight is determined within a few seconds.

The device allows weight to be monitored for several hours. In the event of a power failure, rechargeable batteries secure the necessary power supply.

In addition to the conventional determination of weight, the **seca 985** also has a function for determining body mass index. For this purpose, height is entered on the keypad and the body mass index associated with the weight value is output.

A limit value can be set for use in dialysis centres, with an alarm sounding if this value is not reached.

All the parts can be accommodated on the equipment trolley to allow the scale to be used on a mobile basis.

2. Safety

Before using the new scale, please take a little time to read the following safety instructions.

- Follow the instructions in the instructions for use.
- Keep the operating instructions and the declaration of conformity in a safe place.
- Do not drop the measuring feet or subject them to violent shocks.
- The connecting cables between the measuring feet and the equipment trolley must be routed so that they do not present a tripping hazard.
- Ensure that the mains unit has the right supply voltage (see "Technical data" on page 35).
- Ensure that the cables are not crushed and do not come into contact with hot objects.
- The actuation levers should be folded against the measuring feet once positioning is complete so as to prevent a tripping hazard.
- As a function of its design, the **seca 985** bed scale may be used only on floors which have been produced according to the relevant national standards for the production of floors and floor coverings, with tolerances having been observed. DIN 18202 applies to Germany.
- Have scale serviced and re-calibrated on a regular basis (see chapter entitled "Maintenance/verification" on page 35).
- Have repairs carried out only by authorised persons.

3. Before you start...

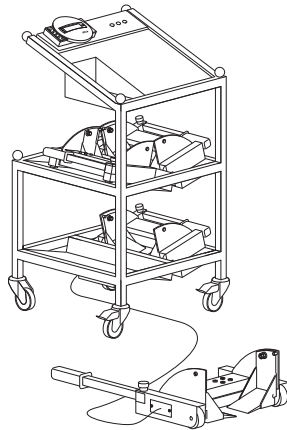
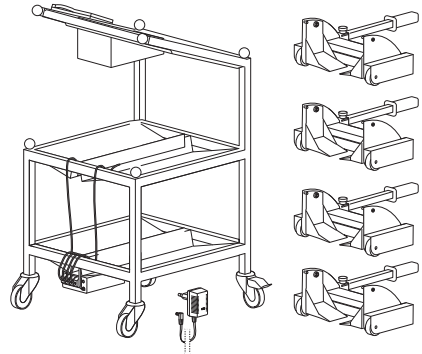
Unpacking

The bed scale is supplied fully assembled.

- Remove the packaging.
- Remove the transport securing devices.
- Lift the equipment trolley from its pallet.

The scope of supply includes:


- 1 trolley
- 4 measuring feet
- 1 mains unit

Charge batteries

The power supply to the scale is usually effected via a mains unit.

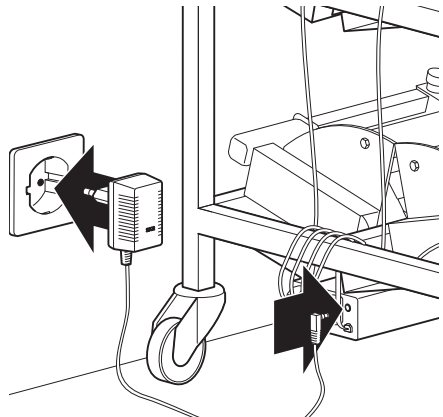
In order to retain values determined, even in the event of a power failure, the bed scale is also equipped with a rechargeable battery. This rechargeable battery is charged by normal operation. You should connect the mains unit to a socket for at least 12 hours in order to be secured against a power failure the first time the scale is used.


Preparation for measurement

The equipment trolley allows the bed scale to be used on a mobile basis, but the scale can also be permanently installed.

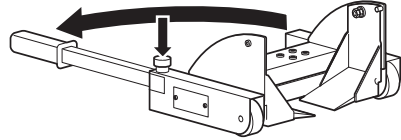- Move the equipment trolley next to the bed.
 - Press the brakes down onto the castors to lock the trolley in position.
 - Remove one measuring foot and place it next to one of the bed castors. Fold out the actuating lever.
- To unlock the lever, press down the round knob.

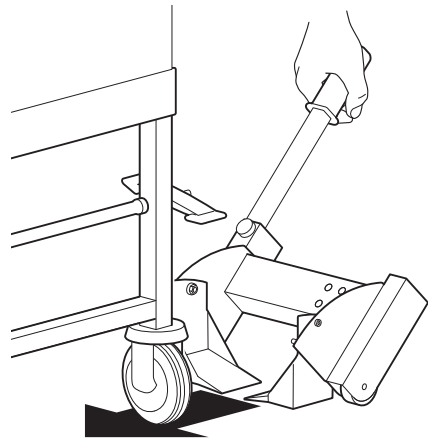
- Keep the actuating lever at an angle of approx. 45° to the floor and push the measuring foot up against the bed. The oscillating arm should now be next to the bed castor.
- Now push the measuring foot further forward – holding the oscillating arm firmly in the other hand. The oscillating arm is now under the bed castor.


- Press the actuating lever downwards. The measuring foot moves under the castor, lifting the bed.

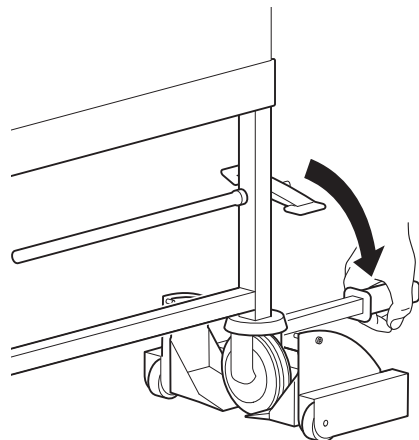The floor under the measuring foot should be level. You must make sure that the connecting cable is not caught under the measuring foot, otherwise faulty measurements may result.

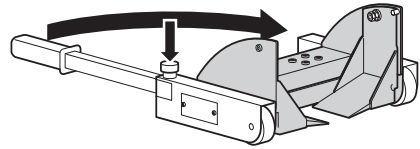- Fold the actuating lever against the measuring foot.
- Proceed in exactly the same way to position the other three measuring feet.


Route cable so that it is not in contact with the grey parts shown in the drawing.

- Plug the mains cable/mains unit into a socket.

If power fails during measuring, power will be supplied by the integrated rechargeable batteries – for at least 4 hours if batteries are fully charged.

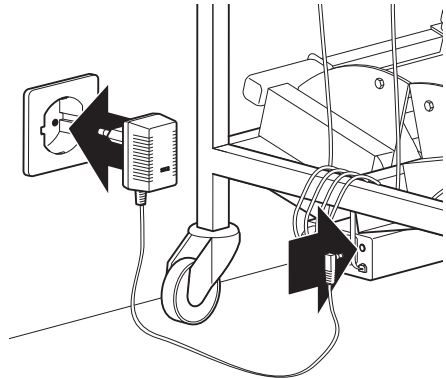
Please use only the mains unit supplied or mains units from the seca range of accessories. Before plugging the unit into the socket, check whether the mains voltage marked on the mains unit matches local mains voltage.

4. Operation

Controls and displays

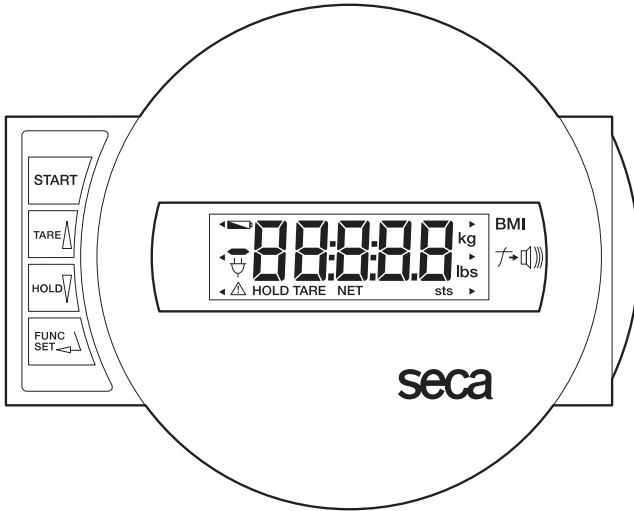The controls and the display are located in the display part.

Switch scale on and off

press briefly → set display to zero, activate/deactivate tare function
in function menu → increase value, scroll

press briefly → activate hold function
in function menu → decrease value, scroll

Activation of (BMI and limit value) functions
Key for entering data, setting damping

Important displays:

Power supplied by mains unit (normal status)

Power supplied by rechargeable batteries (in event of power failure)

Checking that the contents of the calibration counter are correct

This seca scale is calibrated. Calibration may be performed only by authorised centres. To ensure this, the scale is fitted with a calibration counter which records any change to data relevant to calibration. The calibration counter stamp can be found on the display housing.

If you want to check whether the scale is properly calibrated, proceed as follows:

- If necessary, switch off the scale.
- Keep any key depressed and start the scale.

The current contents of the calibration counter is flashing on the display for a few seconds.

- Compare the output contents of the calibration counter with the number shown on the calibration counter sticker.

For valid calibration, both numbers must match. If sticker and calibration counter do not match, then re-calibration is required. Contact your service agent or seca Customer Service.

A trapezoidal button with the word "START" written inside.A rectangular digital display showing the number "2" in a large, bold, black font.A circular sticker with the word "seca" at the top and the number "2" in the center.

Both numbers must match.

Correct weighing

- Press the **START** key. **SECA**, **888888** and **00** appear consecutively in the display. The scale is then automatically set to zero and ready to use.

The plug symbol  appears in the display.

The scale remains in permanent operation until it is switched off.

- Get the patient into bed.
- Read patient weight off the digital display.

If a load greater than 250 kg is placed on the scale, **STOP** appears in the display.

- To switch off the scale, press the **START** key again.

A trapezoidal button with the word "START" written inside.A simple line-art icon of a two-prong electrical plug.A rectangular digital display showing the word "STOP" in a large, bold, black font.A trapezoidal button with the word "START" written inside.

Observing change in weight (TARE)

The Tare function is very useful if the scale is to be used during a dialysis treatment. Patient weight can be tared away – the difference in weight then appears as a positive or negative value in the display.

- Press the green **START** key.
SECA, **00000** and **00** appear consecutively in the display. The scale is then automatically set to zero and ready for operation.
- Get the patient into bed. The patient's weight appears in the display.
- In order to tare the patient's weight away, activate the Tare function by pressing the **TARE** key.

The display is reset to zero. **00** appears in the display. The note "NET" appears.

The scale determines the difference in weight and a positive or negative value appears in the display.

The Tare function can be turned off again by pressing the **TARE** key again. The Tare memory is erased and the current weight appears in the display.

Note:

If the scale is switched on when the patient is already in bed, 00 appears in the display. A slow continuous change in weight, such as typically occurs during dialysis, can be recorded from the 00 display only in the above-described Tare mode. However, the Tare function can be activated only when the display is not 00.

A small trick overcomes this. Adding another load, e.g. a pillow, changes the display and the Tare function can be activated.

Tip:

It makes sense to combine this function with the limit value function (see section entitled "Bottom limit value" on page 31).

Store weight value (HOLD)

You can also continue displaying the weight determined once the load has been removed. This makes it possible to look after a patient first, before having to note the weight.

- Press the **HOLD** key whilst the load is on the scale.

“HOLD” appears in the digital display. The value is stored in the display until the HOLD function or the scale are switched off.

The triangle with the exclamation mark indicates that this frozen display is an additional facility which is not subject to calibration.

- The HOLD function can be switched off by pressing the **HOLD** key again.

00 appears in the display if the scale has no load on it, otherwise the current weight. The “HOLD” display vanishes.

Note:

The HOLD and TARE functions can also be used simultaneously.

Body Mass Index (BMI)

Body Mass Index puts height and weight in relationship to one another, providing more accurate information than ideal Broca weight, for example. A tolerance range is given which is considered ideal for health.

It makes sense to activate the BMI function if current patient weight can be seen in the display.

- Press the **FUNC** key.
The last function used appears in the display.

- Select the BMI function **FU 1** using the arrow keys. Activate the function by pressing the **FUNC** key again.

On the display, you will see the last height entered.

- You can use the arrow keys to adjust the value in increments according to the graduations displayed.

- Once the correct value is set, press the **FUNC** key again. The BMI function is now activated

The arrow indicator points to the symbol for the BMI function.

- Read off the BMI on the digital display and compare it with the categories on page 30.
- Press an arrow key if you would like to see the weight briefly.
- To return to normal weighing mode, press the **FUNC** key.

The arrow indicator for the BMI function goes out.

Evaluating Body Mass Index for adults:

Compare the value determined with the categories quoted below which correspond to those used by WHO, 2000.

- **BMI below 18.5:**
Warning, this patient weighs too little. There could be a tendency to anorexia. An increase in weight is recommended to improve well-being and performance. If in doubt, consult a specialist.
- **BMI between 18.5 and 24.9:**
The patient is of a normal weight. He can stay as he is.
- **BMI between 25 and 30 (pre-obese):**
The patient is slightly to moderately overweight. He should cut his weight if he is already ill (e.g. diabetes, hypertension, gout, fat metabolism disorders).
- **BMI over 30:**
Weight reduction is essential. Metabolism, cardiovascular system and bones are all under strain. A consistent diet, plenty of movement and behavioural training are all recommended. If in doubt, consult a specialist.

Bottom limit value

The **seca 985** bed scale has a limit value function which generates an audible alarm if the display value fails to reach a value which can be set. This allows maximum permissible weight losses in dialysis, for example, to be linked to an alarm.

- The display shows the display value to be monitored. Press the **FUNC** key to set the required limit value.

The last function used appears in the display.

- Use the arrow keys to select function **FU 2**. Activate the limit value function by pressing the **FUNC** key again.

You will see the last limit value set on the display.

- You can change the value in 100 g increments using the arrow keys.

- When the right value is set, press the **FUNC** key again.

The limit value function and the alarm are now activated. The arrow indicator points to the symbol for limit value function.

- You can test the limit value function by lifting the bed so that the measured value fails to reach the set threshold. The alarm sounds. If you release the bed again, the alarm goes silent.

- In order to switch off the limit value function, press the **FUNC** key again.

The arrow indicator for limit value function goes out.

Important note:

if the scale has been switched off, the limit value function must be reactivated as described above.

Weighing with pre-Tare

If the pre-Tare function is activated, a stored weight value is deducted from the currently measured weight. This function is useful, for example, if the weight of a bed is known and is not to be displayed. As a result, the patient can remain in bed throughout the entire weighing process.

Tip:

Make a note of their tare weight on the beds used for weighing - on an adhesive label, for example.

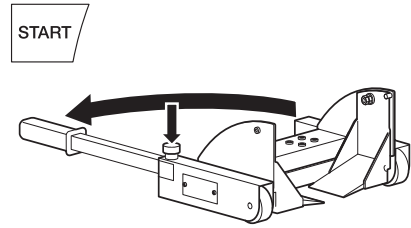
- Prepare for measurement as described in the Section entitled "Preparation for measurement" on page 24, but **do not yet** push the measuring feet under the bed castors.
- Press the **START** key with no load on the scale.
SECA, **88888** and **0.0** appear consecutively in the display. The scale is then automatically set to zero and ready for operation.
- Press the **FUNC** key. The last function used appears in the display.
- Use the arrow keys to select the pre-Tare function, **FU 3**.
- Activate the function by pressing the **FUNC** key again. In the display you will see the tare value from the memory.
- You can use the arrow keys to adjust the tare value in increments according to the graduations displayed.
- Once the correct tare value is set, press the **FUNC** key again. The pre-Tare function is now activated.
- Now gradually push all four measuring feet under the bed castors to lift the bed.
- Read the weighing result off the digital display.
- To return to normal weighing mode, press the **FUNC** key.
- To switch off the scale, press the **START** key again.

Moving and storing the scale

- Switch off the scale.
- Unlock the actuating lever of the measuring foot by pressing down the round knob and folding out the lever.
- Raise the actuating lever to lower the castor.
- Pull the measuring foot out to the side, wind up the cable and place the measuring foot in one of the compartments of the equipment trolley.
- Proceed in exactly the same way to remove the remaining three measuring feet and store them on the trolley.
- Disconnect the mains plug from the socket and likewise store the mains unit on the equipment trolley.
- Release the brakes on the equipment trolley.

You can now move the bed scale to its next point of use or “park” it somewhere out of the way until it is required again.

5. Setting damping

Damping is useful for reducing weight determination malfunctions (due to patient movements, for example). The stronger the damping, the more slowly the weight display reacts. You can set damping to suit requirements.

- Press the **FUNC** key and keep it depressed for 2 seconds.
FIL and the current setting appear in the display:
 - 0 = slight damping
→ fast determination of weight
 - 1 = average damping
→ normal determination of weight
 - 2 = strong damping
→ slow determination of weight.
- Press an arrow key if you want to change the set value.
- Press the **FUNC** key again to store the displayed value. The scale is then back in normal weighing mode.

The settings are retained even once the scale is switched off.

6. Cleaning

Clean the equipment trolley and the measuring feet as required using a domestic cleaning agent or a commercially-available disinfectant. Follow the manufacturer's instructions. Under no circumstances use abrasive or acid cleaners, white spirit, benzene or the like for cleaning. Such substances can damage the high-quality surfaces.

7. What do I do if...

... there is nothing in the display?

- Is the scale switched on?
- Is the plug contact on the scale properly connected??

... **00** does not appear before the weighing operation?

- Start the scale again. Do not change the load on the bed during switching on.
- Check whether the measuring feet are correctly positioned under the bed and that the oscillating arms for the bed castor are freely suspended.

... the scale goes off again after a few seconds?

- The mains unit is not connected. Connect the mains unit to a socket and start the scale again.

... the display shows ?

- The power supply from the mains was interrupted during measuring. If the rechargeable battery is fully charged, measuring continues for at least 4 hours. Restore the power supply as soon as possible.

... **bAtt** appears in the display?

- The rechargeable batteries have run down so far that measuring is no longer possible without a power supply. Restore the power supply.
- If this fault occurs quite frequently, have the rechargeable batteries checked.

... **5tOP** appears in the display?

- Maximum load has been exceeded.

... the display flashes?

- If you have just activated the Hold or Tare function, please wait until the scale has detected a steady weight value. The display will then stop flashing.
- If there has been no activation of a function beforehand, remove the load from the scale and wait until **00** is displayed, then weigh again.

... the display **tEMP** appears?

- The ambient temperature of the scale is too high or too low. Place the scale in an ambient temperature between +10 °C and +40 °C. Wait about 15 minutes for the scale to adapt to the ambient temperature and then weigh again.

... the display $E_r: 111$ appears?

- At least one weighing cell is not correctly connected or the connection is back to front. Check the connections. The colours of plug and socket must match. Switch the scale off using the Start key and restart. The scale then works normally again. If this is not the case, inform Servicing.

... the display $E_r: 1i2$ appears?

- The maximum weight for switching on was exceeded when the scale was switched on, e.g. if bed and patient are being weighed. Remove the load from the scale and start it again. The scale then works normally again. If this is not the case, inform Servicing.

... one segment is permanently illuminated or is not illuminated at all?

- The relevant segment is faulty. Inform Servicing.

8. Maintenance/verification

Have the scale verified by authorised persons in accordance with national legal requirements. Verification is essential if one or more security marks are damaged or the verification counter does not match the number on the applicable verification counter mark (cf. page 27).

We recommend having your scale serviced by your local service agent before having it verified. seca Customer Service will be pleased to give you further assistance.

9. Technical data

Dimensions

width:	520 mm
height:	927 mm
depth:	562 mm
weight	approx. 25 kg

Measuring feet

width:	328 mm
height:	150 mm
depth:	212 mm
weight:	approx. 6.2 kg

Cable length: 2.80 m

Size of figures in display 20 mm

Temperature range +10 °C to +40 °C

Calibration as per Directive 2014/31/EU verified, Class III

Medical product as per Directive 93/42/EEC Class I with measuring function

Maximum load 250 kg

Minimum load 2 kg

Graduations

to 200 kg:	100 g
200 kg to 250 kg:	200 g

Accuracy at first calibration

to 50 kg:	± 50 g
50 kg to 200 kg:	± 100 g
200 kg to 250 kg:	± 200 g

Subtractive Tare device

T- = 250 kg

Capacity

Lim = 500 kg

Maximum tare weight of bed

250 kg

Power supply

9 to 12 V= controlled

10. Spare parts and accessories

Power supply

Model dependent mains unit

Plug-in mains unit, Euro: 230 V~ / 50Hz, 12 V= / 150mA

seca spare part no.: 68-32-10-252

Plug-in mains unit, Int: 100-240 V~ / 50-60Hz, 12 V= / 0.5A

seca spare part no.: 68-32-10-265

Plug-in mains unit

Use only seca original plug-in mains units with 9 V or controlled 12 V output voltage. Commercially-available mains units may supply a higher voltage than stated and thus lead to damage to the scale.

11. Disposal

Disposing of the device

Do not dispose of the device in domestic waste. The device must be disposed of properly as electronic waste. Follow the national regulations which apply in your case. For further information, contact our service department at:

service@seca.com

Rechargeable batteries

Do not throw old rechargeable batteries into the domestic refuse! Ensure that rechargeable batteries are properly disposed of at local collection points.

12. Warranty

A two-year warranty from date of delivery applies to defects attributable to poor materials or workmanship. All moveable parts - batteries, cables, mains units, rechargeable batteries etc. - are excluded. Defects which come under warranty will be made good for the customer at no charge on production of the receipt. No further claims can be entertained. The costs of transport in both directions will be borne by the customer should the equipment be located anywhere other than the customer's premises. In the event of transport damage, claims under warranty can be

honoured only if the complete original packaging was used for any transport and the scale secured and attached in that packaging just as it was when originally packed. All the packaging should therefore be retained.

A claim under warranty will not be honoured if the equipment is opened by persons not expressly authorised by seca to do so.

We would ask our customers abroad to contact their local sales agent in the event of a warranty matter.

13. Declaration of conformity

seca gmbh & co. kg hereby declares that the product meets the terms of the applicable European directives. The unabridged declaration of conformity can be found at: www.seca.com.

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